


To operate the table's power functions*...

Press & hold the appropriate button on the foot / hand control.

* 222 - Base Up / Down only

223 - Base Up / Down & Back Up / Down

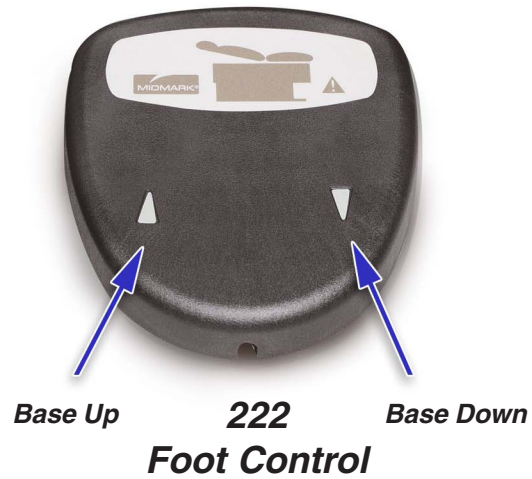

222 only

To manually position the back section...

- A. Squeeze the Back Release Handle (either side).
- B. Move the back section to desired position.
- C. Release handle to lock back section in position.



Equipment Alert

If the table's weight limit is exceeded, you will hear a "beep" and the function will be disabled. If the problem occurs with a normal load, contact your service technician.

Maintenance

Periodically inspect the following:

- Power cord must be free of visible damage.
- All fasteners must be present and secure.
- All mechanical functions should operate.

Cleaning

Upholstery

Wash your upholstery weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner build-up.

Disinfect your upholstery with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners. Follow this with a clear water rinse and thorough drying of material. See current CDC Guideline for Disinfection & Sterilization in Healthcare Facilities.

To minimize damage caused by disinfectant cleaner residue build-up, do not allow disinfectants to pool on the upholstery surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on www.midmark.com in the Technical library under the User Information tab for your product.

Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth, and mild cleaner. Periodic applications of common furniture wax will ease cleaning, and maintain the finished luster of the surfaces.

Calling for Service

If service is required, contact your Midmark dealer.

To contact Midmark directly:
1-800-Midmark (1-800-643-6275)
8:00 am until 5:00 pm Mon-Fri (EST)



- **Français** : Référez-vous à www.midmark.com
- **Deutsch**: Siehe www.midmark.com
- **Español**: Refiera a www.midmark.com

Midmark Corporation
60 Vista Drive
Versailles, OH 45380-0286
Phone: 937-526-3662
Fax: 937-526-5542
midmark.com



Because we care.

Ritter®
by MIDMARK

Quick Reference Guide

222 & 223

Barrier-Free® Exam Table

Model Number:

Serial Number:

Dealer Info:

Purchase Date:

Upholstery Color:

003-1612-00 Rev. E 7/10/15

This guide outlines basic operation of the table. The enclosed User Guide provides additional instructions on operation and care, as well as regulatory compliance information. If service is required, contact your authorized Midmark dealer.



MA7330600p



Drawer Heater (optional)
To turn the drawer heater ON / OFF...
Press the drawer heater switch.
(The switch illuminates when heater is ON).

ON OFF

Exam Assistant® Drawer System

To lower drawer front...

Pull drawer front down to release magnetic latch.



Equipment Alert

The max. weight the shelf will support is 10 lbs (4.5 kg).

If drawer front folds past horizontal...

Pull drawer front upward until it snaps back into the horizontal position.